AT52305

**WÓZEK INWALIDZKI ELEKTRYCZNY INSTRUKCJA UŻYTKOWANIA
ELECTRIC WHEELCHAIR USER MANUAL
GEBRAUCHSANWEISUNG FÜR ELEKTOROLLSTUHL
UŽIVATELSKÁ PŘÍRUČKA K ELEKTRICKÉMU INVALIDNÍMU VOZÍKU
ELEKTRICKÝ INVALIDNÝ VOZÍK – NÁVOD NA POUŽITIE
MANUEL D'UTILISATION DU FAUTEUIL ROULANT ÉLECTRIQUE
GEBRUIKERSHANDLEIDING ELEKTRISCHE ROLSTOEL
MANUAL DE USUARIO DE LA SILLA DE RUEDAS ELÉCTRICA
MANUALE D'USO DELLA SEDIA A ROTELLE ELETTRICA
ANVÄNDARHANDBOK FÖR ELEKTRISK RULLSTOL**

MD

PL

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ENG

Before using your power wheelchair, read the instructions and follow the information, warnings and cautions provided. If this manual contains information that you do not understand, or if you need further assistance with assembly or operation, please contact your authorised dealer.

Strict adherence to the warnings, notices and instructions set out in this manual guarantees safe use of the product. The manufacturer accepts no responsibility for any injury or damage caused by failure to observe the warnings, cautions and instructions for use. Failure to observe the warnings in the instructions for use may result in injury or damage to the power wheelchair.

INTRODUCTION

1 The design of the wheelchair allows it to be easily unfolded and folded, providing extreme convenience when stored or transported in a car boot.

2 This wheelchair is designed for people with disabilities. The maximum weight of the user is 100 kg.

3 This wheelchair is designed for use outdoors, indoors, on flat surfaces, but cannot be used on grass, gravel roads, steep inclines, expressways, in rain or snow.

PRECAUTIONS FOR SAFE USE

- To ensure your safety, we kindly ask you to read the contents of this manual carefully. Special care must be taken when driving the wheelchair.
- Please read the following warning carefully to avoid damage or injury as a result of improper operation. The manufacturer cannot be held responsible for injuries or damage to the product caused by improper use. Be aware that other road users may not be able to see a wheelchair user. The use of safety devices such as reflectors, reflective clothing, lights and safety signage is essential to ensure safety.
- We recommend that you consult your doctor regularly to ensure that your condition allows you to use your wheelchair safely.
- Do not use a wheelchair under the influence of alcohol. Alcohol can adversely affect the safe use of the wheelchair.
- The use of the wheelchair for purposes other than mobility, i.e. weightlifting, sports, athletics, towing, is prohibited. Use other than for the intended purpose may result in a safety hazard and may place undue strain on the wheelchair.
- This wheelchair is intended for use by one person only. It is forbidden to carry passengers or heavy objects. Carrying loads risks the wheelchair tipping over.
- Special care must be taken before entering/exiting the wheelchair to avoid tipping over.
- Do not lean on the armrests when dismounting from the wheelchair, as this risks damaging the product or causing injury.
- Avoid resting your body weight on the footrest. This can cause the wheelchair to tip over and cause injury.
- Using a wheelchair in manual mode without a companion can cause injuries.
- Avoid sitting in a wheelchair by yourself as this may cause injury. You should be assisted by a companion.
- It is forbidden to make changes to the wheelchair without the manufacturer's approval. Adding additional devices/parts may change some of the wheelchair specifications such as overall weight, size, centre of gravity.
- A faulty wheelchair may stop at any time during use. In the event of abnormal or irregular operation, stop using it and contact your dealer.
- Do not switch the wheelchair into manual mode on an incline, as this risks spontaneous, uncontrolled movement of the device.
- When moving, the wheelchair user should ensure that items of clothing and parts of the body are at a sufficient distance from the wheels. Loose clothing and other objects can be pulled into the wheels.
- It is prohibited to connect other devices to the wheelchair's electronic system or battery.
- The wheelchair should be stored at room temperature. Prolonged storage of the product in a low or high temperature environment will adversely affect its functioning.
- Do not use the wheelchair with a depleted battery. Check the state of charge of the battery before each use.
- When turning, slow down; sharp turns are prohibited. When turning, care must be taken to maintain a stable centre of gravity. In order to prevent the product from tipping over, the centre of gravity must not be shifted to the opposite side when turning.
- When entering a slope, head straight up, the wheelchair should move supported by both front wheels at the same time. To reduce the risk of tipping over, the wheelchair should not move up the hill at an angle. Both wheels on either side of the wheelchair should move up an incline at the same time. Special care should be taken when going up any slopes.
- When moving up a hill, the wheelchair should be constantly in motion. If it is necessary to stop, the wheelchair should be slowly increase speed.
- Down hills should be descended at minimum speed. If the speed down the incline is faster than expected, gently release the handlebar lever to stop the wheelchair and then lightly press the lever to control the speed of the downhill movement.
- Reversing is only possible on flat roads. When reversing, you must drive smoothly at low speed. During the course, stop frequently and check that the road there are no obstacles. Do not lean down or backwards along the slope to avoid tipping over.
- Although the wheelchair can negotiate high obstacles, we recommend that you do not overcome heights exceeding 25 mm. Such actions may cause the wheelchair to become unstable. Driving a wheelchair over an obstacle of this height may cause the product to topple over and cause serious injury to the user.
- Do not expose the product to open flames.
- Do not tow a wheelchair.
- Do not negotiate escalators in a wheelchair.
- Do not sit in a wheelchair while in any vehicle.
- Always disconnect the power supply before moving. Make sure that

the brakes are locked to prevent the product from moving during transfer. It is permissible to drive a wheelchair into the lift, but the power must be switched off and the brake applied.

- When transporting, do not place the wheelchair in the front seat of the vehicle; it is recommended to carry the product in the boot.
- Radio wave sources such as radio stations, TV stations, amateur radio transmissions, two-way radios and mobile phones can affect wheelchair control.

1. do not switch on portable personal communication devices such as CB radios and mobile phones near a running wheelchair;
2. care should also be taken to stay away from nearby radio or TV station transmitters.
3. in the event of unexpected movement or release of the brake, the wheelchair should be switched off as soon as possible;

CAUTION: When using and operating the wheelchair and when folding and adjusting the mechanisms, there may be a danger of trapping and/or pinching parts of the user's/companion's body in the openings/gaps between components. Perform these operations with particular care. Once the adjustment has been completed, stabilise the position by carefully tightening the nuts/bolts.

NOTES:

In the event of a product-related 'serious incident' that directly or indirectly led to, could have led to or is likely to lead to any of the following:

- (a) the death of a patient, user or other person; or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious threat to public health

this 'serious incident' should be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. For Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

In the event of pain, allergic reactions or other distressing symptoms unclear to the user related to the use of the medical device, use of the device should be discontinued and a healthcare professional should be consulted.

3. Performance and structure

3.1 Construction of the electric wheelchair

- 1.Push handle 2.Armrest 3.Frame 4.Brake lever 5.Drive wheel 6.Battery 7.Self-adjusting wheels 8.Footrest 9.Seat 10.Backrest 11.Handgrip

INTENDED USE:

The product is designed to comfortably transport people with limited or no walking ability. The product is designed to transport one person.

ATTENTION: it is forbidden to use the product other than for its intended purpose

CONTRAINDICATIONS:

Physical or mental limitations (e.g. visual impairment) that prevent safe handling of the wheelchair.

-Body Balance and Stability Disorders - People who have balance or body stability problems may have difficulty using an electric wheelchair safely. This includes people with advanced forms of neurological or muscular diseases.

-Deficiencies in perception and spatial awareness - Problems with depth perception or spatial awareness can increase the risk of accidents when using an electric wheelchair.

-Insufficient ability to operate controls - Electric wheelchairs require a certain level of manual ability to operate a joystick or other controls. People with severe manual limitations may need specialized adaptations.

-Serious cognitive impairments - Illnesses such as advanced dementia can prevent the safe use of an electric wheelchair, as a person may not be able to respond appropriately to their surroundings or remember how to operate the device.

-Medical issues that require continuous lying down or immobilization -People who need to remain lying down for other severe physical limitations may not be able to use a typical electric wheelchair.

-Lack of ability to sit independently - Individuals who cannot sit independently or maintain a sitting posture due to other problems may require special adaptive solutions.

CHARACTERISTICS OF AN ELECTRIC WHEELCHAIR

- Classification by type of electric shock protection: the wheelchair is connected to an internal power source

- Classification according to the degree of protection against electric shock: the wheelchair belongs to application part type B

- I PXO; classified according to the degree of protection for the inlet liquid

- Safety level classification of flammable anaesthetic gas mixed with air or oxygen or nitrous oxide: equipment other than AP or APG

- Classification by operating mode: continuous operation

- Rated voltage and frequency: 24 V direct current (DC)

- Power supply: internal power supply


- Has the wheelchair been fitted with a discharge protection device: None

- Does the wheelchair have a signal output and an input section: None

- Permanent or non-permanent installation device: mobile device. Wiring of the electric wheelchair

CIRCUIT DIAGRAM

1. Handle
2. Regulator
3. Left engine connectors
4. Battery connectors
5. Right engine connectors
6. 24V lithium battery
7. Right engine
8. Right brake
9. Left brake
10. Left engine

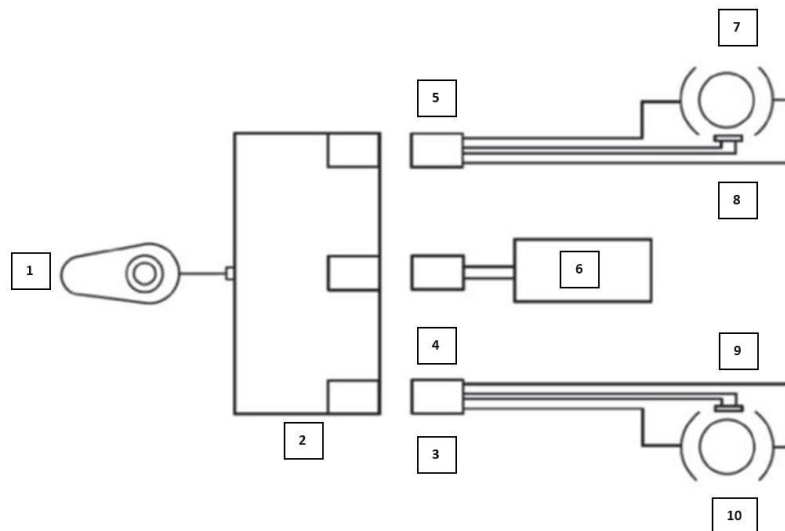Note: The maximum potential range is tested depending on the area, handling, etc.

SCOPE OF SUPPLY

STORAGE

Store the wheelchair in a clean indoor environment with relative humidity <80%, temperature between 0-50°C. Remove the battery before storing the wheelchair. Failure to remove the battery may result in rusting of the frame and damage to the electronics. Avoid prolonged exposure to sunlight.

DISPOSAL OF THE PRODUCT

The wheelchair is an electrical product, please dispose of the equipment in a manner appropriate for the disposal of electrical and electronic equipment.

The battery should be treated in exactly the same way.

Wheelchair, battery pack, joystick, charger, instruction manual.

PRE-LAUNCH PREPARATIONS

Power must be switched off before getting into the wheelchair, only switched on when the user is in the wheelchair. The power must be switched off each time you get down from the wheelchair. Before sitting down, unfold the footrest and hold on to the armrest; if boarding from the side, pull the armrest upwards. When dismounting, it is forbidden to stand on the footrest as this can lead to dangerous tipping over and damage to the product.

NOTE: Before use, check that no component is damaged.

TECHNICAL DATA

Model: AT52305

Dimension when unfolded (L*W*H): 950*625*930 mm

Dimensions when folded: 410*625*775 mm

Weight: 21KG

Actual seat depth: 410 mm

Seat width: 430 mm

Width between armrests: 470 mm

Height of front seating surface: 520 mm

Backrest angle: 10°

Backrest height: 400 mm

Backrest width: 400 mm

Distance from footrest to seat: 465 mm

Distance from footrest to ground: 90 mm

Footrest length: 165 mm

Footrest width: 310 mm

Footrest and leg angle: 106°

Leg angle to seat surface: 117°

Armrest height: 220 mm

Distance from the front of the armrest to the backrest: 340 mm

Armrest length: 265 mm

Armrest width: 50 mm

Armrest angle: 5°

Distance between armrests: 470 mm

Obstacle clearance height: ≥25 mm

Maximum speed: ≤6 km/h

Horizontal stopping distance on the road: ≤1000 mm

Maximum load: 100 kg

Battery: 1.4 kg

Range: 10 km

Maximum angle of elevation: ≥3°

Static stability: >6°

Dynamic stability: ≥3°

Motor specification: 24V 150 W*2

Battery specification: 24V 6Ah*1

Max. regulator output current Brushless: 35A

Max. charger output current: 2A

Front wheel specification: Non-pneumatic tyre, diameter external - 150 mm

Rear wheel specification: Non-pneumatic tyre, diameter external - 270 mm




This symbol indicates the maximum weight of the user

ASSEMBLY, FOLDING AND ADJUSTMENT

1 Remove the product from the packaging and place it on the ground. First unfold the footrests and then fit the battery pack onto the bottom of the battery rail (Figures 1 and 2). Unfold the wheelchair by pulling the backrest with one hand and holding the lower bar with the other hand (Fig. 3). A characteristic "click", indicates that the latch has automatically locked and the wheelchair is ready for use.

2 Insert the controller into the armrest tube (Fig. 4), then tighten the knob to secure the controller.

3. Remove the battery (Fig. 5): Press the switch, then pull out the battery.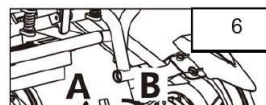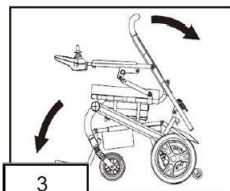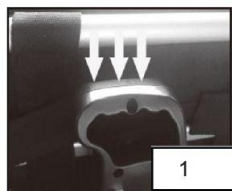


BRAKE LEVER ADJUSTMENT

A set of braking devices is attached to the motors. The wheelchair operates in electric mode when the brake lever is in the A position. In this case, the wheelchair should only be propelled using the drive, it is prohibited to push the wheelchair by hand. Putting the brake lever in position B activates the mode manual, and thus the product must be pushed to move (Fig. 6).

ATTENTION: A wheelchair set in manual mode must only be used with the assistance of an assistant. Using it alone may result in injury. Do not set the wheelchair in manual mode when it is on an incline. If this happens, the wheelchair may roll off on its own, causing injury to the user or the product.

ADJUSTMENT OF ARMRESTS

In order to provide comfort when getting in and out of the wheelchair for people with walking difficulties, it is possible to tilt the armrest upwards. To tilt the armrest, press the locking button underneath and then pull the armrest upwards (Fig. 7). To move the armrest down, press down until you hear a "click" indicating that the armrest is locked. The armrest is now in its original position.

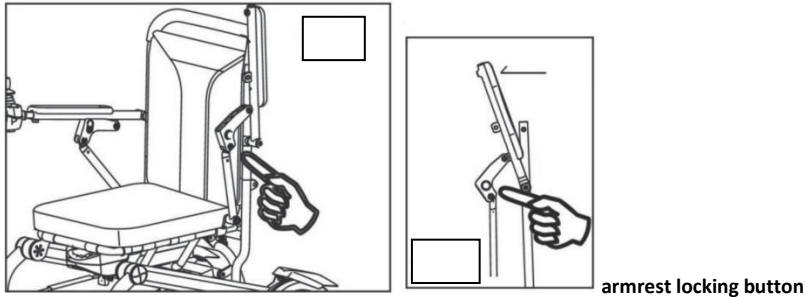NOTE: Tilting the armrest upwards too far will put it into lock mode. To unlock the armrest it must be pressed down firmly.

USE

Joystick: controlling electronic device. (Fig. 9)

The joystick is usually installed on the armrest and connected to the electrical box along with the motor and battery.

- Power switch: the button supplies power to the control system, which then transfers power to the motor. Do not use the switch button to stop the wheelchair except in emergency situations. This method of stopping may adversely affect the life of the wheelchair's drive components.
- Joystick lever: used to control the direction and speed of the wheelchair. Further tilting the lever away from the centre causes the wheelchair to accelerate; movement in the opposite direction causes it to move more slowly. Releasing the lever automatically returns the wheelchair to its initial state and brakes.
- Audible signal button: Pressing the button triggers an audible warning signal.
- Speed decrease button and speed increase button: When the power is switched on, the speed indicator displays the current maximum driving speed. The displayed value will decrease or increase by one bar each time the speed decrease or increase button is pressed. The number of bars displayed represents the current maximum speed. The range can be adjusted by the user.
- Charger socket: The socket is for wheelchair charging only. The socket must not be used as a power source for any other electrical devices. Connecting other electrical devices may damage the control system or decrease the performance of the wheelchair.

JOYSTICK

1. On/off button
2. Speed reduction button
3. Increase speed button
4. Horn
5. Charge indicator
6. Speed indicator
7. Handle

	0-4,5 km/h
	0-4 km/h
	0-3,5 km/h
	0-3 km/h
	0-2,5 km/h

NOTE: If the wheelchair moves unexpectedly, release the lever immediately so that the wheelchair stops automatically.

NOTE: The warranty does not cover damage to the control system caused by connecting a device other than the charger supplied.

USE OF THE BATTERY

A new battery should be fully charged before first use. This allows the battery to reach approximately 90% of its maximum capacity. After each use, the battery should be charged for 4-6 hours and then the wheelchair can be used again. After four or five charging cycles, the battery will reach 100% capacity. If the battery is damaged, it should be completely replaced. In order to protect the environment, the used battery should be disposed of properly. If the battery is not used, it is recommended to carry out more than a 20-minute charging cycle every two months to extend its life. Keep a safe distance from sources of fire and heat when charging. The product should be charged in a free, dry and well-ventilated area away from flammable, explosive or chemical substances.

ATTENTION: If the electric wheelchair has not been used for three days, the safety system automatically puts the battery into sleep mode. To unlock and restart the wheelchair, connect the charger and charge the battery before turning it on.

Type: Lithium-ion battery

Voltage: 24V

NOTE: Do not use wheelchairs when the ambient temperature is below -25 degrees or above 50 degrees.

CHARGING THE BATTERY WITH THE CHARGER

- Make sure the wheelchair is switched off and then connect the battery to the joystick.
- Connect the charger output plug to the 3-hole socket located on the battery on the back of the wheelchair.
- Insert the plug at the other end of the charger into a standard wall socket. When the blue light comes on, it indicates that charging has started and when the green light comes on, it indicates full charging.
- We recommend charging the battery for 4-6 hours.
- Once the battery is fully charged, remove the plug and charger.

Warning! The battery should be charged in a room with a temperature of 0-35°C. Charging outside this range may cause damage to the battery or injury to the user. The product should be charged using the original charger supplied by the manufacturer.


OVERLOAD AND UNDERVOLTAGE PROTECTION

Safeguards are installed on the battery protection board to ensure the safety of the wheelchair. If the motor is overloaded or the voltage of the unit drops, the power supply is switched off. This helps to protect the motor and electrical components from damage. To restore the protective function, recharge the wheelchair using the charger or reconnect the battery plug. (10).

SAFETY BELT

For safety reasons, the seat belt (11) must be fastened when using the wheelchair.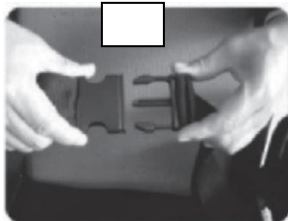

TARGET PATIENT GROUP

Persons suffering from diseases, dysfunctions or injuries for the treatment/rehabilitation or compensation of which this device is intended (see section on intended use of the device in these instructions). The device can be purchased by the user on their own or on the recommendation of a doctor, therapist or other specialist. Whether you purchase the device yourself or on the recommendation of a doctor/therapist/other specialist, you must take into account the available sizes/necessary functions/sizes/variants of the device, the indications and contraindications for use, as well as the information provided by the manufacturer.

Table 1 Diagnosis of errors:

2 warning signals + 1 error signal	Connection error. Check the manoeuvre and regulator.
2 warning signals + 2 error signals	Left motor failure or wrong plug connection
2 warning signals + 3 error signals	Failure of the left brake or incorrect plug connection

2 warning signals + 4 fault signals	Failure of the right motor or incorrect plug connection
2 warning signals + 5 fault signals	Failure of the right brake or incorrect plug connection
2 warning signals + 6 fault signals	Regulator in current overload or motor short-circuited
2 warning signals + 7 fault signals	Handle or lever failure / do not return to position
2 warning signals + 8 fault signals	Regulator failure
2 warning signals + 9 fault signals	Controller or battery failure

NOTE: If the wheelchair malfunctions, release the lever, which causes the wheelchair to stop automatically. Stop using the product immediately and contact an authorised service centre or dealer.

CONSERVATION

The wheelchair requires regular routine maintenance.

- Avoid using the wheelchair in a wet environment. Direct or prolonged exposure to water or moisture can cause electronic and mechanical malfunctions of the wheelchair. Water can corrode electrical components and the seat frame.
- Some parts of the wheelchair are sensitive to temperature. Extreme heat or cold negatively affects the function of the wheelchair
- Avoid applying excessive force to the regulator and especially to the handle.
- Avoid prolonged exposure of the wheelchair to extreme environmental conditions, i.e. excessive heat, cold or humidity.
- Keep the regulator clean.
- Red LEDs lit on the battery indicator indicate that the batteries are almost running low. Recharge the batteries as soon as possible.
- The bearings of all wheels are lubricated and sealed at the factory. No lubrication required
- The battery can also be charged when removed from the wheelchair.

MAINTENANCE AFTER USE

- Switch off the power supply (please disconnect all plugs when the product will not be used for a long time)
- The wheelchair should not be operated by children or people with cognitive impairments.
- If the wheelchair has been soiled, it must be cleaned immediately after use according to the specified cleaning conditions.
- Storing the product at room temperature extends the service life.

CLEANING

- Clean the wheelchair with a damp, clean and soft cloth with very mild detergent and then dry thoroughly. Do not use any chemical cleaning agents (protection against deformation and discolouration).
- Do not use harsh objects, solvents or disinfectants for cleaning.
- If the seat cushion is dirty, it should be washed by hand and then dried completely.
- Never wash the wheelchair with water or expose it directly to water. The surface of the wheelchair frame is covered with a protective coating.
- Details of the weekly, monthly and half-yearly checks are shown in Table 2.

TABLE 2 (MAINTENANCE AND CLEANING)

Elements to be controlled	Each time	Every week	Every month	Every six months
Wheel hubs, drive mechanism and fasteners			<input type="checkbox"/>	
Handle efficiency	<input type="checkbox"/>			
Braking system	<input type="checkbox"/>			
Connections/cables		<input type="checkbox"/>		
Battery status	<input type="checkbox"/>			
Tyre condition			<input type="checkbox"/>	
Inflation status of rear wheels	<input type="checkbox"/>			
Frame condition				<input type="checkbox"/>
Condition of motor and actuator				<input type="checkbox"/>
Condition of the front wheels		<input type="checkbox"/>		
Cleanliness	<input type="checkbox"/>			

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. AN TAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period AN TAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. AN TAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to AN TAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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