# User manual

MANUEL D'UTILISATION  
GEBRUIKSAANWIJZING  
GEBRAUCHSANWEISUNG  
MANUALE DI ISTRUZIONI  
MANUAL DE INSTRUCCIONES  
INSTRUKCJA OBSŁUGI  
NÁVOD K OBSLUZE

## Normandie Plus

# Contents

<b>Preface</b>	<b>4</b>
<b>1. Your product</b>	<b>5</b>
1.1. Options	5
<b>2. Before use</b>	<b>6</b>
2.1. Intended use	6
2.2. General safety instructions	6
2.3. Symbols on the product	7
2.4. Assembly/Disassembly	8
2.5. Transport and storage	9
<b>3. Using your wheelchair</b>	<b>10</b>
3.1. Transfer in and out of the wheelchair	10
3.2. Comfort adjustments	10
3.3. Running wheels / Parking brakes	13
<b>4. Maintenance</b>	<b>14</b>
4.1. Points of maintenance	14
4.2. Maintenance instructions	15
4.3. Expected lifespan	15
4.4. Reuse	15
4.5. End of use	16
4.6. Warranty	16
<b>5. Technical specifications</b>	<b>16</b>

# Preface

Thank you for your trust in the products of Vermeiren. To support you on the use of this wheelchair and its operating options, this manual is offered. Please read it carefully; it will help you to get familiar with the operation, capabilities and limitations of your wheelchair.

If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. He/she will be glad to help you.

## Important note

To ensure your safety, and to prolong the lifetime of your product, please take good care of it and have it checked and serviced on a regular basis.

This manual reflects the latest product developments. Vermeiren has the right to implement changes to this type of product without any obligation to adapt or replace similar products previously delivered.

Pictures are used to clarify the instructions in this manual. Details of the depicted product may deviate from your product.

## Information available

On our website <http://www.vermeiren.com/> you will always find the most recent version of the information in this manual. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

	User manual For user and specialist dealer
	Service manual for wheelchairs For specialist dealer
	EC declaration of conformity

# 1. Your product

1. Base frame
2. Backrest
3. Seat
4. Armrests
5. Arm pads
6. Leg rest
7. Footplates
8. Inclination lever
9. Wheels
10. Push bar
11. Identification plate

## 1.1. Options

Contact your specialist dealer about options. He will gladly advise you.

## 2. Before use

### 2.1. Intended use

In this paragraph a brief description of the intended use of your product is given. Additionally, relevant warnings are added to the instructions in the other paragraphs. In this way we would like to make you aware of the possible misuse that may appear.

- This product is a medical device.
- The wheelchair is intended to be used by the elderly, during revalidation, or by patients suffering from paralysis.
- This wheelchair is suitable for indoor use only, on flat, level grounds.
- This wheelchair is designed and produced solely to transport/transfer one (1) person with a maximum weight of 130kg. It is not designed for transportation of goods or objects, nor for any use other than previously described.
- Only use accessories and spare parts approved by Vermeiren.
- Please read all technical details and limits of your wheelchair in chapter 5..
- The warranty on this product is based on normal use and maintenance as described in this manual. Damage to your product caused by improper use or lack of maintenance will cause the warranty to lapse.

### 2.2. General safety instructions

#### CAUTION

##### **Risk of injuries and/or damage**

- Please read and follow the instructions in this manual. Otherwise you may get injured, or your product may get damaged.

Be aware that some parts of this wheelchair may get very hot or cold due to ambient temperature, solar radiation, heating devices. Be careful when touching. Use protective clothing if the weather is cold.

Be careful when using possible causes of fire such as cigarettes since they may set the seat and back covers alight.

Do not modify your product in any way.

Any serious incident [MDR (EU) 2017/745 §2 (65)] that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

### 2.3. Symbols on the product

	Maximum weight of the user in kg
	Maximum safe slope in ° (degrees).
	For indoor use only
	Type designation
	Catalogue number
	Serial number
	Medical device
	Manufacturer
	Date of manufacture
	Declaration of conformity
	Attention: important information
	It is advised to read the manual

## 2.4. Assembly/Disassembly

Upon delivery of the wheelchair, the seat and backrest have to be installed before use. The seat and backrest can also be removed for easy transport.

### 2.4.1. Backrest

1. Place the backrest tubes over the frame tubes.
2. Press the spring button **(1)** to slide the backrest down.
3. Push the backrest all the way down until it is secured into place.

To remove the backrest, press the spring button beneath the upholstery and pull the backrest upwards until it detaches from the frame.

### 2.4.2. Seat


1. Place the hooks on the rear side of the seat over the frame tube close to the backrest.
2. Push down on the front side of the seat to fix it into place. You will hear a click.

To remove the seat, pull on the front of the seat until it detaches from the frame.

### 2.4.3. Footplates

1. Insert the footplate tube **(2)** into the attachment **(3)** on the back of the leg rest. Make sure the footplates are located on the correct side of the leg rest.
2. Tighten the footplate tube into place by inserting the screw **(4)** and tightening it. To insert the screw, make sure that the footplates are at a 90° angle to the leg rest.

3. Repeat for the other footplate.

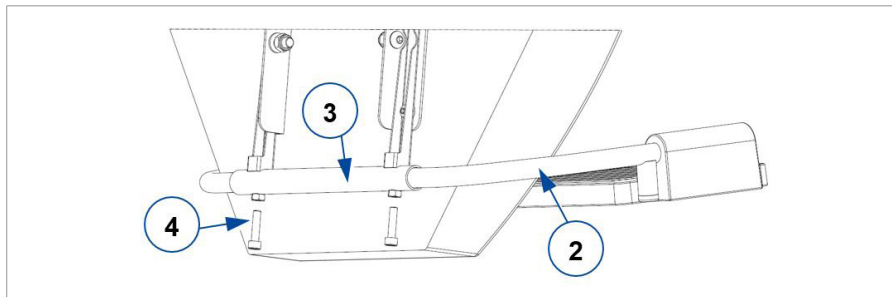## 2.5. Transport and storage

### **⚠ CAUTION**

#### **Risk of injuries and damage**

- Pay attention when passing with the wheelchair through narrow passages.
- Make sure the passage is clear from objects, obstacles and persons.
- Do NOT use your wheelchair as a seat in a vehicle, see the following symbol.

### 2.5.1. Storage

### **⚠ CAUTION**

#### **Risk of damage**

- Make sure that your wheelchair is stored in a dry environment to prevent mould from growing and the upholstery from being damaged, see also chapter 5..

# 3. Using your wheelchair

## **⚠ WARNING**

### **Risk of injuries**

- First read previous chapters and inform yourself about the intended use. Do NOT use your wheelchair unless you have read and fully understood all instructions.
- In case of doubts or questions, do not hesitate to contact your local specialist dealer, your care provider or technical adviser to help you with this.

## 3.1. Transfer in and out of the wheelchair

### **⚠ CAUTION**

#### **Risk of injuries or damage**

- In case you cannot perform the transfer in a safe manner, ask someone to assist you.
1. Check that the brakes of the wheelchair are engaged.
  2. Place the wheelchair in its most upright sitting position.
  3. Transfer to/out of the chair. Place the arm pad at the same height as the seat cushion for a sideways transfer (see paragraph **3.2.2.**).

### 3.1.1. Correct position in the wheelchair

Sit down on the seat with the lower back against the backrest.

## 3.2. Comfort adjustments

### **⚠ CAUTION**

#### **Risk of injuries or damage**

- Always apply the brakes before adjustment.

### 3.2.1. Backrest inclination

#### **⚠ CAUTION**

#### **Risk of injuries or damage**

- Be sure that the wheelchair does not tip over backwards when it is placed in its most backwards position.
- Make sure that no objects or individuals are in the adjusting area (see entrapment zones).

The angle of the backrest and the leg rest can be adjusted by means of a gas spring.

Entrapment zones are indicated in the figure below by **(1)**.

To incline the comfort chair:

1. Engage the brakes, see **§3.3.**
2. To incline the backrest, pull the lever **(2)** and push against the backrest until you reach the desired position. Let go of the lever to block the backrest into position. The backrest and leg rest incline simultaneously. (Backrest inclination: 13° - 45°; leg rest inclination: 0° - 90°)
3. To reduce the backrest and leg rest inclination, pull the lever **(2)** and let the backrest and leg rest return to the desired position. Let go of the lever to block the backrest and leg rest into position.

### 3.2.2. Adjusting the armrests

The armrests can be adjusted in height and can be lowered to the height of the seat frame for the sideways transfer of the patient.

1. Pull the lever of the armrests **(3)** upwards (in the direction of the arm pad).
2. The locking spring will be released and the armrest can be moved within the armrest socket **(4)** into the desired position (range 114 mm, steps of 19 mm).

#### **⚠ CAUTION**

##### **Risk of injuries**

- If the locking spring is guided outside the armrest socket **(4)**, the armrest is not locked. Use this position only for the sideways transfer of the patient.

For sideways transfer of the patient:

- Pull the lever **(3)** and guide the armrest down to the lowest possible position. Now the arm pad comes at the same height of the seat cushion.

### 3.3. Running wheels / Parking brakes

#### **⚠ CAUTION**

#### **Risk of injuries or damage**

- Lock both rear wheels in place to put the wheelchair in a parking position.
- Make sure the wheelchair is on a flat horizontal surface before releasing the brakes. Never release both brakes simultaneously.

	<p><b>5</b> Engaging the parking brakes</p> <p>With your foot, press the brake plate of the wheel downwards until it locks into place. The wheels cannot roll.</p>
	<p><b>6</b> Disengaging the parking brakes</p> <p>With your foot, press the brake plate of the wheel in the direction of the frame. The wheels have now been released and can both roll and swivel.</p>

# 4. Maintenance

- i** Regular care ensures that your wheelchair is preserved in a perfectly functional condition. For the maintenance manual, refer to the Vermeiren website: [www.vermeiren.com](http://www.vermeiren.com).

## 4.1. Points of maintenance

### **⚠ CAUTION**

#### **Risk of injuries and damage**

- Repairs and replacements may only be undertaken by trained persons and only genuine replacement parts of Vermeiren should be used.
- i** The last page of this manual contains a registration form for the specialist dealer to record each service.  
The service frequency depends on the frequency and intensity of use. Contact your dealer to agree to a common timetable for inspection/maintenance/repair.

## **Before each use**

Inspect the following points:

- All parts: Present and undamaged or unworn.
- All parts: Clean, see **§ 4.2.1.**
- Wheels, backrest, seat: Well secured.
- Working of gas springs, brakes, armrests.
- Wheels/tyres; keep them free of wires, hair, sand, fibres and other contamination (oil, mud).
- Condition of frame parts: No deformation, instability, weakness or loose connections
- Upholstery, seat cushion, calf-rest cushion: No excessive wear (like dented spots, damage or tears).

Contact your specialist dealer for possible repairs or part replacements.

## 4.2. Maintenance instructions

### 4.2.1. Cleaning

#### CAUTION

##### **Risk of damage**

- Never use a steam cleaner to clean the chair.

Wipe all rigid parts of the wheelchair with a damp cloth (not drenched). If necessary, use a mild soap, suitable for varnishes and synthetics.

The upholstery can be cleaned with lukewarm water and a mild soap. Do not use abrasive cleaning agents to clean.

### 4.2.2. Disinfection

#### CAUTION

##### **Risk of damage**

- Disinfections may only be undertaken by trained persons. Consult your specialist dealer.

## 4.3. Expected lifespan

The average lifespan of your wheelchair is 5 years. Depending on the frequency of use, driving circumstances and maintenance, the lifespan of your wheelchair will increase or decrease.

## 4.4. Reuse

Before each reuse, have the wheelchair disinfected, inspected and serviced according to the instructions in **§ 4.1.** and **§ 4.2.**

## 4.5. End of use

At end of life, you need to dispose your wheelchair according to the local environmental legislation. The best way to do so, is to disassemble the wheelchair to facilitate the transport of recyclable parts.

## 4.6. Warranty

The warranty on this product is subject to the general terms and conditions of each country.

# 5. Technical specifications

The technical details below are only valid for this wheelchair, at standard settings and optimal ambient conditions. Take these details into account during use. The values are no longer applicable if your chair has been modified, damaged, or is severely worn.

<b>Brand</b>	<b>Vermeiren</b>
<b>Product group</b>	<b>Manual wheelchair</b>
<b>Type</b>	<b>Normandie Plus</b>
<b>Description</b>	<b>Dimensions</b>
Maximum user weight	130 kg
Overall length	1110 mm
Overall width	685 mm
Total mass	Approx. 31,4 kg
Mass of heaviest part	21,2 kg
Masses of parts that can be dismantled or removed	Seat: 2,85 kg Backrest: 7,1 kg
Seat plane angle	8°
Effective seat depth	480 mm
Effective seat width	500 mm
Seat surface height at front edge	520 mm
Backrest angle	13° - 45°
<b>We reserve the right to introduce technical changes. Measurement tolerance +- 15 mm / 1,5 kg / 1,5°</b>	

<b>Brand</b>	<b>Vermeiren</b>
<b>Product group</b>	<b>Manual wheelchair</b>
<b>Type</b>	<b>Normandie Plus</b>
<b>Description</b>	<b>Dimensions</b>
Backrest height	750 mm
Armrest height	0 mm (only for sideways transfer) 150 mm - 210 mm
Overall height	1205 mm
Dismantled height	490 mm
Thickness seat cushions	100 mm
Angle between seat and leg rest	0° - 90°
Rated slope	15°
Storage and use temperature	+5°C - +41°C
Storage and use humidity	30% - 70%
<b>We reserve the right to introduce technical changes. Measurement tolerance +- 15 mm / 1,5 kg / 1,5°</b>	

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## Service registration form

This product (name): .....

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:

Vermeiren Group  
Vermeirenplein 1/15  
2920 Kalmthout  
Belgium  
[www.vermeiren.com](http://www.vermeiren.com)

Version: A, 2024-06

Basic UDI-DI: 5415174NormandiePlusYU

All rights reserved, including translation.

**VERMEIREN**  
we care for you