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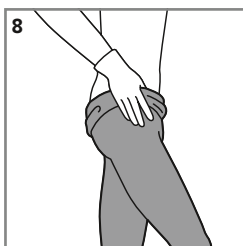
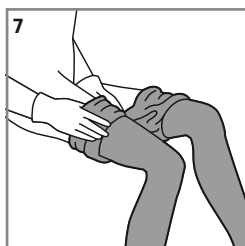
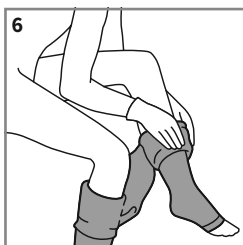
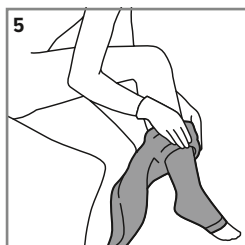
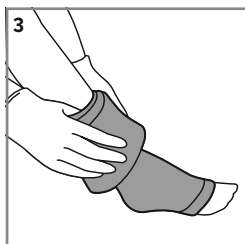
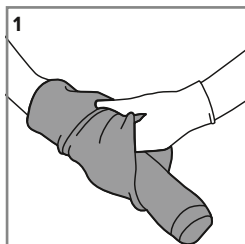
EN

Intended purpose

Compression therapy involves measures to apply extensive, external pressure to treat vein problems and lymph drainage disorders. The aim of the extensive pressure is to prevent the formation of oedema and to support venous return and lymph drainage. Compression garments exert an optimal pressure gradient that gradually decreases from bottom to top. Compression therapy is primarily used to treat vein problems and oedema. It involves the use of compressive and elastic double-stretch fabrics or knitted fabrics that wrap right round

the extremities. As a general rule, compression therapy is a long-term treatment requiring appropriate products that are used by the patients themselves when medically indicated and on the instructions of a doctor. Zipper extension included, choking hazard. Please store out of the reach of children – parents are responsible for their children.

Target group of users and patients

Potential users include patients and healthcare professionals. The target group of patients includes adults, adolescents and children who are treated independently by medical professionals based on the relevant measurements, available sizes and optional extras, as well as the intended purpose, indications and contraindications – always in accordance with the manufacturer's instructions.

Notes for use

For compression stockings/tights with open toes, use the Juzo Slippie donning aid. Your specialist retailer will be happy to show you the correct way to use the product. You can also visit juzo.com/donning for video instructions relating to all Juzo donning aids. Put your compression garment on the decongested leg in the morning right after getting up. Put on and put of your compression garment while sitting down to prevent injuries or falling. Rough, unfilled finger nails or sharp-edged rings etc. may damage the very fine knitted fabric. We therefore recommend the use of Juzo special gloves. Toe nails and calluses can also cause damage, as can the defective inner lining of shoes. In order to increase the wearing comfort of your compression garments, you can hoist the silicone border a little (if there is one) a number of times during wearing. Please make sure that with combined treatments, a sufficient overlap between treatments is guaranteed.

Compression stockings

1. Pull the stocking over your forearm and hold on to the foot section from the inside with your fingers. Turn the stocking inside out by pulling it out as far as the heel.
2. Using both hands, stretch open the opening and pull the foot section over your foot as far as the instep. Make sure that the heel fabric is sitting on the heel.
3. Gradually pull the stocking further over your foot – as you do so, you will also turn it back the right way out.
4. Now pull up the stocking evenly and free from any wrinkles – without twisting it or pulling at the edges – and at the same time apply it to your leg with the flat of your hand. If you have pulled the stocking up too high, simply push it back down to the ankle and then pull it up again with less strength. Tip: fold over the silicone border before donning (if applicable).

Compression tights

5. Turn inside out as far as the heel, slip in the first leg as described in steps 2 and 3 and fit to below the knee.
6. Repeat the procedure with your other leg.
7. Once you have put the tights on both legs to below the knees, turn both sides the right way out again and pull them up both legs bit by bit. Then adjust the sit of the knitted fabric with the flat of your hand.

8. Now pull the body part up to your waist. Make sure that the gusset is positioned vertically in the crotch. Please bear in mind that your compression tights must never be overstretched. Distribute the knitted fabric in stages from the bottom to the top.

Please note:

Only wear compression garments if they have been prescribed by a doctor. Compression garments should only be dispensed by properly trained staff at a medical retailer. Your Juzo compression garments are largely resistant to oils, ointments, skin moisture and other environmental influences. The durability of the knitted fabric may be affected by certain combinations of the above. This is why it is advisable to have your compression garments checked regularly at a medical retailer. For compression garments with a silicone border, please note: the relevant part of the body should be depilated and free from oils, powder, creams and ointments. Please contact your specialist retailer in the event of damage. For hygiene reasons, this product is intended for one person only. The label on this product is necessary for identifying and tracing the product. We therefore ask you not to remove the labels under any circumstances. Where zipper extension is included: choking hazard. Please store out of the reach of children – parents are responsible for their children.

Washing and care instructions

Please be sure to follow the care instructions on the product label. Do up the fasteners (if there are any) before washing. In the washing machine, it is recommended to use a laundry bag. Turn compression garments inside out before washing, and if possible, wash them separately or with articles of clothing in the same colour. Wash your compression garment every day after wearing. Your compression garment can be washed in the washing machine on a gentle or delicate wash cycle at 40 °C. Make sure to use a detergent for delicate fabrics (without a fabric softener). We recommend you use our gentle Juzo special detergent for hand-washing your garment. When hand-washing, rinse well and do not wring. You can reduce drying time by placing the compression garment on a thick towel after washing, tightly rolling the towel up and firmly pressing out excess moisture. Afterwards, the compression garment can be hung up loosely. Do not leave the garment in the towel. Do not place your compression garment on a radiator or in direct sun. If your compression garments are suitable for tumble drying (see product label), please use a gentle cycle. Juzo compression garments are not suitable for dry cleaning.

Material composition

Please refer to the product label for detailed information.

Storage information and shelf life

Store in a dry place and protect from sunlight. Standard products have a maximum shelf life of 36 months. The compression garment then has a usage period of 6 months. The usage period depends on correct handling (e.g. care, donning and doffing) and normal use of the products. The usage period of the medical product is printed on the box with an hourglass symbol. Custom-made products are for immediate use. Body measurements may change based on the patient's condition

and individual circumstances. Custom-made products are therefore designed for a usage period of 6 months. After the 6-month usage period, body measurements must be re-checked by a medical specialist. In the event of significant changes in measurements of the body part that is being treated (e.g. based on therapeutic success, weight change), re-measurement and supply of a new product may be required earlier, before the 6-month usage period comes to an end.

Indications

Chronic venous diseases: Improvement of venous symptoms, improvement of quality of life in chronic venous diseases, prevention and therapy of venous oedema, prevention and therapy of venous skin changes, eczema and pigmentation, dermatoliposclerosis and atrophie blanche, therapy for venous leg ulcer, therapy for mixed (arterial and venous) leg ulcers (taking into account contraindications), prevention of venous leg ulcer recurrence, pain reduction in venous leg ulcers, varicosis, initial phase after varicose therapy, functional venous insufficiency (in obesity, sedentary occupations, standing occupations), venous malformations

Thromboembolic venous diseases: Superficial venous thrombosis, deep vein thrombosis, condition after thrombosis, post-thrombotic syndrome, thrombosis prophylaxis in mobile patients

Oedemas: Lymphoedema, oedema during pregnancy, post-traumatic oedema, post-operative oedema, post-operative reperfusion oedema, cyclic idiopathic oedema, lipoedema, congestion due to immobility (arthrogenic congestion syndrome, paresis and partial paresis of the limb), occupational oedema (standing occupations, sedentary occupations), oedema due to medication if no change is possible

Other indications: Obesity with functional venous insufficiency, inflammatory dermatoses of the legs, nausea, dizziness in pregnancy, congestive disorders in pregnancy

In case of pregnancy, changing abdomen circumferences mean that a pregnancy body part should be exclusively used for treatment with pantyhose.

Following burns and for scar treatment, we recommend using our special scar therapy products.

Contraindications

Advanced peripheral arterial disease (if one of these parameters applies: ABPI < 0.5, ankle artery pressure < 60 mmHg, toe pressure < 30 mmHg or TcPO₂ < 20 mmHg instep). Decompensated heart failure (NYHA III + IV), septic phlebitis, phlegmasia cerulea dolens.

In the following cases, the decision about treatment should be made considering the benefits and risks as well as choosing the most suitable compression option:

Pronounced weeping dermatoses, intolerance to the compression material, severe sensory disorders of the extremities, advanced peripheral neuropathy (e.g. diabetes mellitus), primary chronic polyarthritis.

Important

If there are open wounds in the area to be treated, the affected area must be covered with an appropriate wound dressing or bandage before the compression garment can be applied.

Liability for failure to comply with these contraindications cannot be accepted by the Julius Zorn GmbH Group.

Liability

In case of improper use (including independent repairs or repairs by third-party providers), the manufacturer cannot guarantee the medical device's conformity, which causes the manufacturer's liability to become void.

Residual risks and side effects

If used improperly, rare but undesirable side effects may occur, such as skin necrosis and pressure damage to peripheral nerves. People with sensitive skin may experience itching, flaking and signs of inflammation when wearing compression garments. This is why adequate skin care is necessary when using compression therapy.

The compression garment should be removed immediately if any of the following symptoms occur, and the patient should be evaluated by a doctor: Blue or white discolouration, abnormal sensations and numbness, increased pain, shortness of breath and profuse sweating, acute movement restrictions.

If you are intolerant to one or more of the substances used in this product, please consult your doctor before use. The manufacturer does not accept any liability for damage/injuries resulting from improper use or misuse.

In the case of complaints related to this product, such as damage to the knitted fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

As a rule, the product is to be destroyed via the conventional disposal route after the end of use. Please observe the applicable local/national regulations.

Additional information for products containing silver

As silver can deduct heat and electricity, please avoid the direct contact with electric sources, Juzo Expert Silver/Juzo Expert Strong Silver/Juzo Dynamic Silver should not come in contact with metals (e.g. metal clothes hangers) when the textile is wet, the worked in silver could be discoloured thereby, the tanning agents used in the manufacture of the leather/man-made shoe lining may cause discoloration of the stocking and the silver thread in the foot area.

Additional information about the Dip Dye Collection

Each Juzo compression product with a Dip Dye design is unique. The colour gradient of the individual items is unique and cannot be reproduced exactly. Owing to the manufacturing process, slight colour variations are possible.

Additional information about the Batik Collection

Each Juzo compression product with a Batik design is unique. The colour pattern of the handmade individual items is unique and cannot be reproduced. There may be minor colour changes after the first wash. The characteristic "wrinkled look" originates from the production technique. It is only visible on

the unworn product. Owing to the special colouring technique, these compression products are slightly longer before they have been worn than products without a Batik design. However, this does not have any influence on the fitting accuracy and the medical efficacy when worn.

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Julius Zorn GmbH · Juliusplatz 1 · 86551 Aichach · Germany · info@juzo.com · juzo.com
Berro AG · Austrasse 7 · 9055 Bühler · Suisse
UK responsible person: Juzo UK Ltd