Allevia

030123

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EN Medical device: **ALLEVIA - prophylactic orthotic insoles with asymmetrical metatarsal pads.**

Application of the insoles: Forefoot overload syndrome (transverse flat feet); metatarsalgia, including Morton neuroma; toe deformities; calluses and corns on the plantar side in the area of the 2nd, 3rd and 4th metatarsal heads; hollow foot; rheumatoid foot. **Effect of the insoles (metatarsal pad):** they positively change the distribution of pressures on the plantar side of the feet in the area of the 2nd, 3rd and 4th metatarsal heads; cushion the contact of the metatarsal heads with the ground; prevent the dropped metatarsal heads; reduce the risk of formation of fixed convex forefoot and hammer toes; reduce pain in the area of the metatarsal heads. **Contraindications:** The insoles should not be used by persons with diabetic foot syndrome. **Warnings:** The insoles should not have direct contact with the skin of the foot. When using the insoles, protect your feet by means of socks, knee-highs or stockings. In the case of any doubt related to the application of the device, please consult your physician, physiotherapist or podologist.

Materials used: Natural leather easily absorbing sweat excreted by the feet; textile material lining the metatarsal pad made of latex foam; layer of latex foam on the plantar part. **How to select the insoles to match the user's feet:** The efficiency of the insoles depends on the proper location of the metatarsal pad. The front edge of the pad should reach the place where the 2nd, 3rd and 4th metatarsal heads turn into the shafts. In view of the different anatomical structure of the feet of potential users, the size of the insoles should be selected by a properly trained employee of a shop/podology salon. **Instructions for use and cleaning of the insoles:** The insoles are to be used in shoes with a heel height not exceeding 4 cm. Placing the insoles inside the shoes results in reducing their internal dimensions (fitting and height of the toes). Therefore, the insoles are fit, first of all, for wider fit shoes, with replaceable insoles and for shoes with regulated fastenings in the forefoot and on the instep. Persons who have not used the insoles with the metatarsal pad before may feel uncomfortable at the beginning. It is necessary to get used to the insoles gradually (first day - 1 hour of use, second day - two hours, etc.). If the sense of uncomfortableness is not gone after two weeks, visit the shop or salon where the insoles were purchased. Remove the sweaty insoles from the shoes and let them dry at the room temperature, far from any sources of heat. To clean and take good care of the insoles, you may use e.g. *Avocado Lotion* by Kaps. The insoles should not be used in the case of their deformation and/or significant sweating of the external layer made of natural leather. The worn insoles should be placed into containers for mixed waste. Each serious incident related to the device should be reported to the producer and competent authority of the Member State being the place of residence of the user. **Date of issue of the instructions for use:** version III, 23.02.2022.

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