medi

igli carbon orthotic

Gebrauchsanweisung. Instructions for use. Инструкция по использованию. Modo de empleo. Instrukcja użytkowania. Instruções de utilização. Istruzioni per l'uso. Návod k použití. Használati útmutató. 使用方法. Mode d'emploi. Gebruiksaanwijzing. Brugsanvisning. Bruksanvisning. Инструкции за употреба. Arahan penggunaan. Упатство за употреба.

medi. I feel better.

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Instructions for Use for igli Carbon Orthotics by medi

igli orthotics are aids that have been developed for feet with special requirements.

Intended purpose of foot orthotics

medi foot orthotics (medical devices) for the functional care/treatment of foot deformities.

Indications

Orthotics can be implemented for functional therapy for the following diagnoses:

- **Heel pain / Heel spur / Plantar fasciitis**
igli Heel light / igli Heel spur light, igli Heel / igli Heel spur
- **Knee arthrosis / Knee osteoarthritis**
igli Comfort, igli OA, igli Control
- **Hallux limitus/rigidus**
igli Control
- **Hallux valgus**
Igli Allround, igli Allround Light C+, igli Junior, igli Junior Blue C+, igli Comfort, igli Active, igli Active Light
- **Pes cavus**
Igli Allround, igli Allround Light C+, igli Junior, igli Junior Blue C+, igli Slim, igli Business Silver (W/M), igli Comfort, igli Active, igli Active Light, igli Control
- **Pes valgus**
Igli Allround, igli Allround Light C+, igli Junior, igli Junior Blue C+, igli Slim, igli Business Silver (W/M), igli Comfort, igli Active, igli Active Light, igli Control
- **Fallen arches**
Igli Allround, igli Allround Light C+, igli Junior, igli Junior Blue C+, igli Slim, igli Business Silver (W/M), igli Comfort, igli Active, igli Active Light, igli Control
- **Flat foot**
Igli Allround, igli Allround Light C+, igli Junior, igli Junior Blue C+, igli Slim, igli Business Silver (W/M), igli Comfort, igli Active, igli Active Light, igli Control
- **Splay foot**
Igli Allround, igli Allround Light C+, igli Junior, igli Junior Blue C+, igli Slim, igli Business Silver (W/M), igli Comfort, igli Active, igli Active Light, igli Control
- **Metatarsalgia e.g. Morton's neuroma**
igli Control, igli OA
- **Patellar tendinitis**
Igli Allround, igli Allround Light C+, igli Junior, igli Junior Blue C+, igli Slim, igli Business Silver (W/M), igli Comfort, igli Active, igli Active Light, igli Control

Contra-indications

igli carbon orthotics are contraindicated for diabetes and neuropathy.

Features of the product

igli carbon orthotics are individual orthotics for functional therapy. They constitute an orthotic system for embedding, supporting and stabilising your foot and have a corrective effect which ensures individual adjustment to the patient.

The system consists of three parts:

- Bed (1)
- Carbon frame (2)
- Posts (3)

• Cushion

The orthotics embed the foot so that it is rested, protecting it from pressure peaks and relieving load on individual parts of the foot in a targeted manner.

• Support

The supportive function of the orthotic can relieve load on specific parts of the foot, improve rolling of the foot and helps to stabilise the ankles.

• Correction

The corrective function of the orthotics is achieved via various supportive elements positioned at specific points on the foot, e.g. a corrective support for the arch.

• Pain relief

The structure of the orthotic and the combination of functional materials can help to reduce the pain.

The corrective/support function is provided by the reinforced core (made of carbon in this orthotic) and can be further increased using the posts.

Embedding properties of the posts:


A predominantly embedding effect is achieved by ensuring form-fitting and complete surface contact with the arch of the foot. Two posts, fitted next to each other, are the perfect solution for this. A larger surface area means that less pressure is exerted on the foot.

**Stabilising properties of the posts:**

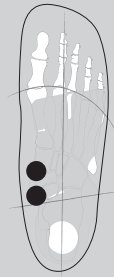In order to stabilise the hindfoot, we recommend that a wedge post is used on the outside of the foot in addition to a ball post on the arch. This provides balanced stability on both sides.

**Corrective properties of the posts:**

Incorrect positioning and biomechanical patterns of motion can be positively influenced by setting the posts in a targeted manner, thus correcting your gait.

Ball post

Wedge post

Important Notes**Instruction for use:**

- Orthotics should always be worn as a pair. Only apply the product to intact or wound treated skin, not directly to injured or damaged skin and only under prior medical guidance. Furthermore, the orthotic is intended to be used multiple times by one individual patient. The products must be issued by experts or trained personnel. Furthermore, the product must be used in accordance with the manufacturer's instructions for use (in accordance with the indication, instructions for putting on, storage and care instructions) and the respective orders from the specialist personnel.
- The stress that the orthotics are placed under due to general wearing, salves, skin moisture and environmental influences can reduce their service life and impair the medical effectiveness of the orthotics. Therefore, we recommend that your doctor and/or specialist retailer checks the orthotics regularly.

Intended users:

- Healthcare professionals and patients, including persons who play a supporting role in care, are the intended users for this device, provided that they have received briefing on this from healthcare professionals.

Intended patient target group:

- Healthcare professionals should provide care to adults and children, applying the available information on the measurements/sizes and required functions/indications, in line with the information provided by the manufacturer, and acting under their own responsibility

Disposal:

- Dispose of properly after use.

Side effects:

- Sometimes, pressure points can be created or the orthotic can slip in the shoe. Therefore wear well-fitting shoes, otherwise using the orthotic will incur risks.

Instructions for putting the orthotics on

- The orthotics can be individually adjusted to the foot and shoe by an expert.
- If required, remove any orthotics that are already in the shoe in order to achieve an optimal fit.
- Some individuals may need to allow their feet to get used to the orthotics. This can last several days - up to two weeks.
- If pain and/or discomfort occur whilst wearing the orthotics, remove the orthotics immediately and contact your doctor or your specialist retailer.
- If you have experienced ongoing problems due to wearing your orthotics, please consult your doctor.
- Please wear the products with hosiery (e.g. socks/stockings/tights etc.).

Instructions for care

- If the orthotics need cleaning, clean them with a damp cloth.
- Do not use any solvents containing acetone to clean them.
- Do not dry your orthotics on the radiator, do not expose them to other direct sources of heat.

Information on the warranty, guarantee and liability

- The product is intended to be used multiple times by one patient and is therefore not intended to be handed over to third parties. If the orthotic is used to treat more than one person or is altered in an incorrect manner, the product liability will be rendered null and void.
- No claims on the guarantee may be asserted for any damages which are caused by improper usage. Discolouration of the orthotics does not constitute the grounds for a complaint.
- The product identification mark on the orthotic is an important component of the product. If you remove this identification from a medi product, the claim to a guarantee, repair or exchange is rendered null and void.
- In the event of any complaints regarding the product such as damage to the orthotic or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state.

The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR).

Transportation and storage conditions

- The products must be stored in a dry place and out of direct sunlight.
- Orthotics have a service life of 42 months. This includes the storage life and the length of time that they may be used for. The length of time that they may be used for is from 6 - 12 months, depending on how much and how heavily they are worn.

The date of the last revision of the instructions is stated on the last page of the instructions. Information on the manufacturer, the conformity symbol and, if required, the authorised representatives, is stated on the packaging.

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